

OPTX-20/20[®] HYDROTAC[™] READING LENSES

Turn **any** eyewear into reading glasses.



OPTX-20/20[®] lenses are small, half-moon shaped reading lenses made of ultra-thin, pliable plastic designed to provide instant magnification to any glasses - safety glasses, sun-glasses, prescription glasses, face shields, etc.

These amazing lenses cling snugly to the inside of any lens, remaining in position until you wish to remove them. They are completely removable and reusable, leaving no messy residue.

OPTX-20/20[®] can be positioned anywhere on your glasses. They are designed to provide distortion-free, edge-to-edge magnification, so they may be trimmed to fit smaller lenses without altering the magnification. Available in 1.5, 2.0, 2.5, and 3.0 magnifications.

- **Removable and reusable**
- **Distortion-free magnification from edge to edge**
- **Can be trimmed to fit any lens**
- **Available in four magnifications**




Stick on top for overhead jobs.


Stick on bottom for reading.



1 Wash glasses and OPTX-20/20[®] with liquid soap and warm water.

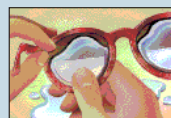

2 With clear water on the inside of the glasses lens, press one OPTX-20/20[®] lens onto the bottom 25% of each side of glasses.

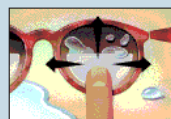

3 Check for proper placement by putting glasses on and looking through OPTX-20/20[®]. Adjustments can be easily made by sliding OPTX-20/20[®] lenses while still wet.



4 When properly adjusted, pat dry with a lint-free towel.

DCT DURABLE COMPONENT TECH

1-800-285-7965

P.O. BOX 15204, ST. LOUIS, MO 63110

OPTX•20/20[®] HYDROTAC[™] READING LENSES

ORDERING INFORMATION

MAGNIFICATION	UOM	SKU
1.5	3/PK	42448
2.0	3/PK	42449
2.5	3/PK	42450
3.0	3/PK	42451